

Base Washing Solution

Liquid. Store at +2/+30°C. For in Vitro Diagnostic Use (IVD). Do not freeze.

Ref No	Package	Ref No	Package
ARBWS	1 x 1 L	ARBWS01	1 x 2 L
ARCBSW	1 x 1 L	ARBWS02	1 x 5 L

INTENDED USE

Base Washing Solution is intended for in vitro diagnostic use in washing cuvettes using the chemistry analyser.

ACTIVE COMPOSITION

Sodium hydroxide.....3,6%

PREPARATION

Base Washing Solution is ready for use.

STABILITY AND STORAGE

Unopened Base Washing Solution packages are stable at +2/+30 °C until the expiration date on the label. The onboard stability of opened Base Washing Solution packages is 90 days. Dispose of products whose onboard stability has expired by removing them from the analyzer.

TEST PRINCIPLE

Base Washing Solution should be used as specified in the analyzers user manual

WARNINGS AND PRECAUTIONS

IVD: For in Vitro Diagnostic use only.

Do not use expired reagents.

Reagents with two different lot numbers should not be interchanged.

For professional use.

Follow Good Laboratory Practice (GLP) guidelines.



H290: May be corrosive to metals.

H314: Causes severe skin burns and eye damage.

P234: Keep only in original container.

P264: Wash hands thoroughly after handling.

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P301+P330+P331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

P303+P361+P353: IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P310: Immediately call a POISON CENTER or doctor/physician.

P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P390: Absorb spillage to prevent material damage.

P501: Dispose of contents and container in accordance with all local, regional, and national regulations.

SYMBOLS

IVD

In Vitro Diagnostic Medical Device

LOT

Lot number

GTIN

Global Trade Item Number

REF

Reference Number

FOR USE WITH

Identifies products to be used together

PRODUCT OF TURKEY

Product of Turkey



Manufacturer



Expiration date



Temperature limitation



Humidity Limitation



Consult instructions for use



Archem Sağlık Sanayi ve Tic. A.Ş.

Mahmutbey Mah. Halkalı Cad. No:124 Kat:4

Bağcılar/İstanbul/Turkey

Tel: + 90 212 444 08 92

Fax: +90 212 629 98 89

info@archem.com.tr www.archem.com.tr

