

## CH WASH

Liquid. Store at +2/+30°C. For in Vitro Diagnostic Use (IVD). Do not freeze.  
 Produced for Siemens Atellica CH930.

| Ref No | Package   |
|--------|-----------|
| BC720  | 2 x 1.5 L |

### INTENDED USE

CH Wash is intended for in vitro diagnostic use in washing cuvettes using the Atellica chemistry analyser.

### ACTIVE COMPOSITION

Sodium hydroxide.....3,6%

### PREPARATION

CH Wash is ready for use.

### STABILITY AND STORAGE

Unopened CH Wash packages are stable at +2/+30 °C until the expiration date on the label. The onboard stability of opened CH Wash packages is 90 days. Dispose of products whose onboard stability has expired by removing them from the analyzer.

### TEST PRINCIPLE

CH Wash should be used as specified in the Atellica analyzers user manual

### WARNINGS AND PRECAUTIONS






IVD: For in Vitro Diagnostic use only.  
 Do not use expired reagents.  
 Reagents with two different lot numbers should not be interchanged.  
 For professional use.  
 Follow Good Laboratory Practice (GLP) guidelines.



**H290:** May be corrosive to metals.  
**H314:** Causes severe skin burns and eye damage.  
**P234:** Keep only in original container.  
**P264:** Wash hands thoroughly after handling.  
**P280:** Wear protective gloves/protective clothing/eye protection/face protection.  
**P301+P330+P331:** IF SWALLOWED: rinse mouth. Do NOT induce vomiting.  
**P303+P361+P353:** IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.  
**P310:** Immediately call a POISON CENTER or doctor/physician.

**P305+P351+P338:** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.  
**P390:** Absorb spillage to prevent material damage.  
**P501:** Dispose of contents and container in accordance with all local, regional, and national regulations.

### SYMBOLS

|  |   |
|--|---|
| <b>IVD</b>   | In Vitro Diagnostic Medical Device      |
| <b>LOT</b>   | Lot number                              |
| <b>GTIN</b>  | Global Trade Item Number                |
| <b>REF</b>   | Reference Number                        |
| <b>FOR USE WITH</b>  | Identifies products to be used together |
| <b>PRODUCT OF TURKEY</b>   | Product of Turkey                       |
|   | Manufacturer                            |
|   | Expiration date                         |
|   | Temperature limitation                  |
|   | Humidity Limitation                     |
|  | Consult instructions for use            |



**Archem Sağlık Sanayi ve Tic. A.Ş.**  
 Mahmutbey Mah. Halkalı Cad. No:124 Kat:4  
 Bağcılar/İstanbul/Turkey  
 Tel: + 90 212 444 08 92  
 Fax: +90 212 629 98 89  
 info@archem.com.tr www.archem.com.tr

