

CLEANER

Liquid. Store at +15/+25°C. For in Vitro Diagnostic Use (IVD). Do not freeze.

Ref No	Package
CLNR01	1 x 1 L

INTENDED USE

Cleaner is a solution used for washing the pipe and probe systems in the device.

SUMMARY

Cleaning with cleaner is necessary to preserve the integrity of the probes and pipes. The cleaner is used to prevent potential contamination from probes and pipes.

ACTIVE COMPOSITION

Compositions:

HCl0.3 mol/L

REAGENT PREPARATION

Reagent is ready for use.

REAGENT STABILITY AND STORAGE ⁴

Reagents are stable at +15/+25°C till the expiration date stated on the label which is only for closed vials.

TEST PRINCIPLE

The cleaner solution should be used as specified in the user manual of the recommended analyzers. The devices it can be used on are Cobas Integra 400 plus, Cobas Integra 800 and Cobas c111.

WARNINGS AND PRECAUTIONS

IVD: For in Vitro Diagnostic use only.

Do not use expired reagents.

Reagents with two different lot numbers should not be interchanged.

For professional use.

Follow Good Laboratory Practice (GLP) guidelines.

Caution:

H290 : May be corrosive to metals.






Precaution:

P234 : Store in its original packaging.

Response:

P390 : Absorb spilled solution to prevent material damage.

SYMBOLS

IVD	In Vitro Diagnostic Medical Device
LOT	Lot number
GTIN	Global Trade Item Number
REF	Reference Number
FOR USE WITH	Identifies products to be used together
PRODUCT OF TURKEY	Product of Turkey
	Manufacturer
	Expiration date
	Temperature limitation
	Humidity Limitation
	Consult instructions for use



Archem Sağlık Sanayi ve Tic. A.Ş.

Mahmutbey Mah. Halkalı Cad. No:124 Kat:4

Bağcılar/İstanbul/Turkey

Tel: + 90 212 444 08 92

Fax: +90 212 629 98 89

info@archem.com.tr www.archem.com.tr

