

MR BUFFER SOLUTION

Liquid. Store at 5°C~35°C. For in Vitro Diagnostic Use (IVD). Do not freeze.

Ref No	Package
MR760	2 x 2 L

Changes made in the instructions for use are marked as grey.

SUMMARY

MR Buffer solution applies to Mindray BS series auto chemistry analyzers. It is used to dilute samples before they are disposed to the electrodes flowcell.

Use the MR Buffer Solution as instructed by the operator's manual of the analyzer.

APPLICABLE INSTRUMENTS

MR Buffer Solution applies to BS series chemistry analyzers manufactured by SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO.LTD.

SPECIFICATION

2 x 2L

COMPONENTS

Components: Formaldehyde: <1.0 w/v%
 Phosphoric acid: <1.0 w/v%


PRODUCT SPECIFICATIONS

1. Appearance: MR Buffer Solution should be clear and transparent liquid without precipitate, suspension and floccule.

REAGENT PREPARATION

Reagents are ready to use.

REAGENT STABILITY AND STORAGE

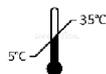

(protect from light)

MR Buffer Solution is stable when stored at 5 °C-35 °C at least until expiration date printed on the label. The buffer vial should be kept away from light. After each use, promptly replace the cap and ensure the vial is capped tightly.

PRECAUTION

MR Buffer Solution is intended for **in vitro diagnostic** use only by trained personnel.

HOW TO USE THIS PRODUCT

Use this product on the chemistry analyzer as instructed in the instrumental manual.

LIMITATIONS

The product only applies to the ISE module of BS series chemistry analyzers. In addition, the warranted results only come from processing tests strictly as instructed in the instrument manual and storing the product correctly.

WARNINGS

1. This product should not be used past the expiration date.
2. Do not use the reagent if it is frozen. If any appearance change occurs, like a cloudy or turbid appearance shows up, do not use this vial of the product any more.
3. Wear proper personal protective equipment (e.g. gloves, lab coat, etc.) and follow safe laboratory procedures when handling this product in the laboratory.
4. This product must be handled by skilled/trained medical professionals.
5. Preservative contained. Do not swallow. Avoid contact with skin and mucous membranes.
6. Dispose of any discarded material in accordance with the requirements of your local government regulations.
7. Material safety data sheet is available for professional user on request.

REFERENCES

Not applicable.

SYMBOLS	
IVD	In Vitro Diagnostic Medical Device
LOT	Lot number
GTIN	Global Trade Item Number
REF	Reference Number
FOR USE WITH	Identifies products to be used together
PRODUCT OF TURKEY	Product of Turkey
	Manufacturer
	Expiration date
	Temperature limitation
	Consult instructions for use



Archem Sağlık Sanayi ve Tic. A.Ş.
Mahmutbey Mah. Halkalı Cad. No:124 Kat:4
Bağcılar/Istanbul/Turkey
Tel: + 90 212 444 08 92
Fax: +90 212 629 98 89
info@archem.com.tr www.archem.com.tr

