TÜRKÇE (TR)

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ENGLISH (EN)

For use with BioRad Variant™ II Turbo A1C

WASH/DILUENT SOLUTION

Always keep this package insert on hand and carefully read it before use.

INTENDED USE:

These products are used in BIORAD VARIANT™ II TURBO series devices and are designed to detect hemoglobin A1c (HbA1c) in whole human blood. Use this product to dilute whole blood samples and wash the tubes through which the sample flows. For use in in vitro diagnostics only.

SUMMARY AND PRINCIPLE:

Stable HbA1c measurement is very useful in the diagnosis and treatment of diabetes and in monitoring diabetic conditions. VARIANT™ II TURBO measures HbA1c in the blood using reversed phase cation exchange chromatography. The blood sample diluted with the hemolysis wash solution is sent to the column that separates the sample into various hemoglobin components based on high performance liquid chromatography (HPLC).

WARNINGS AND PRECAUTIONS

Warnings:

- This Wash/Diluent Solution contains less than 0.02% mucosal- irritative sodium azide (NaN3) as a preservative.
- Sodium azide may also react with copper or lead whereby potentially generating explosive metal azide.
- Sodium azide may also react with acid leading to the generation of toxic explosive hydrogen azide.
- Therefore, when these materials are disposed of, you should flush them with an adequate volume of water in order to prevent any residual reagent in the water servicepipe.

- If these products come into contact with the eyes or mouth, wash the contact area with plenty of water and consult a doctor.
- If it makes contact with skin, wash with plenty of water.
- Discard liquid waste than contains blood sample in accordance with local regulations for biohazardous waste.

Precautions:

- For in vitro diagnostic use only.
- These products are only for VARIANT™ II TURBO series. DO NOT attempt to use with other instruments
- If these products are stored in a cold place, such as a refrigerator, allow them to adjust to the same environment as the instrument for at least one hour before placing them on the instrument.
- DO NOT USE Wash/Diluent Solution that has expired.
- The expiration dates are written on both the box and packlabel.
- Please place each package of Wash/Diluent Solution in the designated place on the bottle tray.

Attach the appropriate syringe package caps to the Wash/Diluent Solution package.

- If the wrong cap is attached to the wrong package, the products may be mixed and cause inaccurate measurements.
- Even if a small amount of Wash/Diluent Solution remains, replace it with a new pack.
- Adding new Wash/Diluent Solution to the old pack may cause inaccurate measurement results.
- Always handle these Always handle Wash/Diluent Solutions with care. with care.
- Avoid physical shock.

MEASUREMENT PROCEDURE
Refer to the operating manual for VARIANT™ II TURBO series.

STORAGE:

- Store unopened Wash/Diluent Solutions packs at a temperature between 15°C and 30°C, avoiding direct sunlight.

- Once opened, use the Wash/Diluent Solutions within one month, even if it is before expiration date
- If the instrument is not to be used for used for one week or more, remove the Wash/Diluent Solutions packs from the instrument. Then, cap them tightly and store them at a temperature between 15°C and 30°C in order to prevent liquid degradation and/or concentration.

MATERIALS PROVIDED:

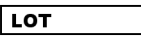
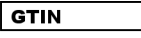
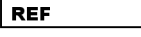
WASH/DILUENT SOLUTION: 2500 mL X 4 packs/ box

Materials of the pack:

Pack with PP (polypropylene) cap. The pack consists of polyethylene (PE) and polyethylene terephthalate (PET).

 Archem Sağlık Sanayi ve Tic. A.Ş.
Mahmutbey Mah. Halkalı Cad. No:124 Kat:4
Bağcılar/İstanbul/Türkiye
Tel: +90 212 444 08 92
Fax: +90 212 629 98 89
info@archem.com.tr www.archem.com.tr

SYMBOLS	
	In Vitro Diagnostic Medical Device
	Lot Number
	Global Trade Item Number
	Reference Number
	Identifies Products to Be Used Together
	Product of Turkey
	Manufacturer
	Expiration Date
	Temperature Limits
	Consult Instructions for Use