

ZINC CONTROL LEVEL I-II

En

REF 06R71-10

REF 06R71-11

FOR USE WITH
ARCHITECT

Package insert instructions must be carefully followed. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this package.

INTENDED USE

Zinc Control Level I-II are for quality control of the Zinc Liquid assay.

CONTENTS / MATERIALS PROVIDED

Zinc Control Level I

REF: 06R71-10

Package: 5 x 3 mL Liquid

Zinc Control Level II

REF: 06R71-11

Package: 5 x 3 mL Liquid

For use with:

Zinc Reagent

Zinc Calibrator

06R71-10 / 06R71-11 do not contain the human serum.

CONTROL STABILITY

Temperature-Conditions	Stability
Unopened at +2/+8°C	Expiry date on the vial.
Opened at +2/+8°C	45 days

PREPARATION OF CONTROL

Control is ready for use.

INDICATIONS OF INSTABILITY OR DETERIORATION

Presence of extreme turbidity or microbial growth may indicate deterioration.

PRECAUTIONS



All human-based products should be handled in accordance with Good Laboratory Practice (GLP) principles using appropriate precautions. Safety data sheets are available at www.archem.com.tr or you may contact your local representative.

WARNINGS

IVD: For in Vitro Diagnostic use only.

Do not use expired controls.

Reagents with two different lot numbers should not be interchanged.

For professional use.

Follow Good Laboratory Practice (GLP) guidelines.

CAUTION: Human source samples are processed with this product. All human source samples must be treated as potentially infectious materials and must be handled in accordance with OSHA standards.

Danger

H317 :May cause allergic skin reaction.

Precaution

P280 :Use protective gloves / clothes / glasses / mask.

P264 :Wash your hands properly after using.

P272 :Contaminated work clothes should not be allowed to be used outside of the workplace.

Intervention

P302+P352 :Wash with plenty of water and soap if it contacts with skin.

P333+P313 :Seek medical help if it irritates your skin or develops rash.

P362+P364 :Remove contaminated clothes and wash properly before using.

Disposal

P501 :Dispose the vials and contents according to the local regulations.

ZINC CONTROL LEVEL I-II

En

REF 06R71-10

REF 06R71-11

FOR USE WITH
ARCHITECT

REFERENCES

1. Burtis CA, Ashwood ER, Bruns DE, editors. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 4th ed. St. Louis , MO, Elsevier Saunders; 2006:2263.
2. Occupational Safety and Health Standards: bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register. July 1, 2001; 17:260–273.
3. Directive 2000/54/EC. Official Journal of the European Communities No. L262 from October 17, 2000.
4. Tietz, N. W. (1983) Clinical guide to Laboratory Tests, p384 W.B. Saunders Co., Philadelphia.

TRADEMARKS







Zinc Control Level I-II are trademarks of ARCHEM Sağlık Sanayi ve Tic. A.Ş. in various jurisdictions.



Archem Sağlık Sanayi ve Tic. A.Ş.
Mahmutbey Mah. Halkalı Cad. No:124 Kat:4
Bağcılar/Istanbul/Turkey
Tel: + 90 212 444 08 92
Fax: +90 212 629 98 89
info@archem.com.tr www.archem.com.tr



SYMBOLS

IVD	In Vitro Diagnostic Medical Device
LOT	Lot Number
CONTROL	Control
GTIN	Global Trade Item Number
REF	Reference Number
FOR USE WITH	Identifies Products to Be Used Together
PRODUCT OF TURKEY	Product of Turkey
	Manufacturer
	Expiration Date
	Temperature Limits
	Consult Instructions for Use
	Caution
	Number of Tests